




Steel

Acidic Special Cleaner

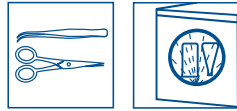

Acidic cleaner for healthcare instrument washers and instruments. Removes limescale deposits, rusty stains and oxide films.

Application

A concentrate intended for cleaning of limescale deposits, stains of rust and oxide films from instrument washing machines and stainless steel instruments. Can be used in immersion baths (manually), in washer-disinfectors and in ultrasound washers.

For professional use.

Product properties

Steel removes deposits and brightens metal surfaces. Targeted for stainless steel, but also suitable for most aluminum, brass and copper materials and removes stains from silicone and plastic surfaces.

Instructions for use

Cleaning of instrument washing machines: 100-300 ml (0.5-1.5% dilution), decalcifying program. If other washing programs are used, ensure that the program does not use other detergents at the same time. Instruments can be washed in the machine during the cleaning cycle.

Cleaning instruments: 20% dilution (2 dl/0.8 l water) in warm water, soaking time from 30 min to 1 h. If there are any deposits, wipe the surface. Wash and rinse instruments right after soaking.

Dilution in an ultrasound washer: 20-200 ml / 1-0.8 l warm water. Soak instruments for 30 min. Recommendation for duration of the program: 10 min. Rinse carefully.

Note

Change the used dilution daily. Acidic product. Do not mix with products containing chloride. To avoid corrosion, do not let the dilution dry on the surface.

Composition

Ingredient		Effect
Organic acids	15-30 %	Deposit dissolvent
Glycolether	5-25 %	Dissolvent, increases washing power
Non-ionic surfactants	< 5 %	Surface moisturizer, active detergent

Product pH

1.5 strongly acidic

Solution pH

3.0 acidic

Storage

Storage temperature under +28°C. Shelf life 3 years from manufacturing.

Appearance and scent

Clear yellowish liquid, acidic scent

Country of manufacture	FINLAND			
User and environmental safety	Serious incidents in relation to Steel acidic detergent are to be reported to asiakaspalvelu@kiilto.com and the local medical device authority.			
	Read the label and product information before use.			
	Safety instructions of the product safety data sheet.			
Package size / Product code	Sales unit	Product Code	GTIN	GTIN (sales unit)
	6 x 1 l	8224	6417964082241	6417964482935



Additional information	Check the updated version of a paper brochure on our website at www.kiilto.com/en or send an e-mail to our customer service at asiakaspalvelu@kiilto.com .
------------------------	--