




## Erinox Special Cleaner

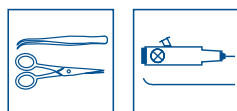

A low foaming detergent for washing instruments and utensils manually and in ultrasonic baths.

**Application** Manual washing of flexible and rigid endoscopes, delicate and sensitive instruments in healthcare and dental care, washing of sensitive glassware and laboratory apparatus, as a detergent in ultrasonic washers of all sizes.

**Product properties** Gentle for use with sensitive instruments and materials, yet removes efficiently grease and body fluids. Efficient cleaner also for instruments and apparatus with narrow lumens.

**Instructions for use** Prepare Erinox special cleaner solution according to the instruction. Change a fresh Erinox dilution daily. In case of manual washing, prepare a new solution for each wash. Rinse the instruments with clean water after washing and dry them. Adjust the instruments loosely into the ultrasonic washer. Rinse the instruments after washing. Disinfect the instruments after washing when needed.

| Instructions for use and dilution % |       | Erinox    | Water  | Notice  |
|-------------------------------------|-------|-----------|--------|---|
| Ultrasonic washer                   |       | 10 -20 ml | 1 l    | Change the dilution 1 - 2 / day<br>Presoaking 2 % if needed |
| Manual washing                      |       |           |        |   |
| Endoscopes                          | 2 %   | 20 ml     | 1 l    |   |
| Other instruments and utensils      | 1-2 % | 10-20 ml  | 1 l    |   |
| Heavily soiled instruments          | 5 %   | 50 ml     | 950 ml |   |

| Composition | Ingredient                 | Effect   |
|-------------|----------------------------|--|
|             | Non-ionic surfactants < 5% | Washing power                                    |
|             | Polycarboxylate <5%        | Washing power, dissolves dirt, binds it to water |

**Product pH** 10.0 weakly alkaline

**Solution pH** 10.0 weakly alkaline

|                                      |   |                     |               |                          |
|--------------------------------------|---|---------------------|---------------|--------------------------|
| <b>Storage</b>                       | > 5 °C, shelf life 3 years from manufacturing.  |                     |               |                          |
| <b>Appearance and scent</b>          | Clear liquid  |                     |               |                          |
| <b>Country of manufacture</b>        | FINLAND   |                     |               |                          |
| <b>User and environmental safety</b> | <p>Serious incidents in relation to Erinox special detergent are to be reported to asiakaspalvelu@kiilto.com and the local medical device authority. Safety instructions of the product safety data sheet.</p> <p>The packaging is made of PE-HD plastic, which is an environmentally friendly, fully recyclable material that may be disposed of by burning.</p> |                     |               |                          |
| <b>Package size / Product code</b>   | <b>Sales unit</b>   | <b>Product Code</b> | <b>GTIN</b>   | <b>GTIN (sales unit)</b> |
|                                      | 6 x 1 l   | 8113                | 6417964081138 | 6417964483536            |
|                                      | 3 x 5 l   | 8114                | 6417964081145 | 6417964483543            |
|                                      | 1 x 200 l   | 8148                | 6417964081480 |                          |
| <b>Additional information</b>        | Check the updated version of a paper brochure on our website at <a href="http://www.kiilto.com/en">www.kiilto.com/en</a> or send an e-mail to our customer service at asiakaspalvelu@kiilto.com.  |                     |               |                          |