

Product overview:

1. LCD display
2. High class sensors to weigh
3. Vibration protection
4. Measuring height
5. A music player inside

Technical specification:

1. Maximum capacity: 20kg
2. Division: 5g
3. Minimum weight: 100g
4. Unit change: kg / lb
5. The range of return to zero: <10kg
6. Low battery indication: a "Lo" at the screen and then automatically power-off
7. Overload indication: a "o-Lo" at the screen when the weighing baby is over 20.10 kg
8. Make zero overrun indication: when you turn on the scale, if it is not in a stable situation, the make zero function will not work, the screen will display "UnST"
9. Supply voltage: 6V with 4*AA batteries
10. Automatically power-off: If there is no operation for over 60 seconds, it will automatically power-off
11. Function keys: 3 keys, ON/OFF/TARE, UNIT, SET
12. Backlight display: blue

Function for keys:

1. **ON/OFF/TARE key**
 - 1) Shutdown situation: press this key to turn on
 - 2) Power-on situation: press this key to tare off
 - 3) Power-on situation: hold the key to power-off

2. UNIT key

In weighing state, you can hold this key 3 seconds to change the unit

3. SET key

In weighing state, press the SET key to check the last number you have weighed. 3 seconds later, it will turn into the weighing situation automatically, keep pressing this key to check the history numbers with 10 groups P0-P9

Specific using function:

1. Tare-off situation:

1) put something less than 10 kg like blanket, press the ON key to start, after a 0.000 kg, take off the blanket and then it will display a minus number in LCD, press the TARE key to clear.

2) Press the ON key to start, after the 0.000kg display, you can put something like a blanket on it, LCD will display the weight of this blanket, as the numbers keep stable, press this key again to clear, take the things off, then a minus number display, pressing it a third time will make it zero again. If you put something on it when it is a minus number, this number will increase.

2. Weighing function:

Put the scale at a horizontal hard ground or table, turn on the scale, after seeing 0.000kg put the baby on the scale at the center, after the number locked, the music will play.

3. Height measuring gauge:

There is a plastic thing that has a 150 cm flexible tape at right side of the scale; you can use this tape to measuring height of your baby. After using this, press the plastic button to put the tape away.

4. Music ON/OFF

Music ON: While in stable situation, turn on the scale and then weigh, after the number locked, you will hear a music singing. Press the middle key once to change to the next song. There are 20 songs for your choice.

Music OFF: While the scales is off, hold SET key (the 3rd key) and then hold ON/OFF key once. You will see "On" in display and then press the UNIT key(the 2nd key), it will become "Off", which means you have turn the music player off.

Simple and easy to use:

1. Open the battery cover and put batteries in
2. Put the scales on a horizontal, hard ground or table, lay a blanket on it (do not let the blanket touch the ground for it will have some influences over weighing)
3. Press the ON key to start, and then read 0.000kg
4. Put your baby on the scale
5. After the numbers locked and the music start, you can pick up the baby
6. No operations in 60 seconds, it will automatically power-off

Attention please:

1. Put the baby at the center for precision weigh.
2. If there is anything irregular, please get off the battery, wait for about 10 minutes then start again
3. When there is a Lo at the start, remember to change batteries (choosing alkaline batteries will be better for the scale)
4. Keep the scale cleaning; do not use corrosive liquids to clean
5. Don't put things that are 1.5 times heavier than its capacity and keep it on a horizontal plane, big crash is not allowed.
6. The batteries should take off if it is not that frequent use



Parc des Forges - 8 rue Louise Michel - F-67200 STRASBOURG - France